
CowMotion

CowMotion can be quickly and easily attached and adjusted to the cattle claw. The time in the hoof stand is shortened and the stress factor on the animal is reduced. For the farmer, veterinarian and hoof trimmer, this means more efficiency and more relaxed work. Basically, our cattle orthoses can be glued with known claw adhesives. We recommend an adhesive with an open time of 60 seconds so that you have enough time to apply the orthosis and seal it to the cap. It is best to use our CowMotion Hoof Glue.


Note!

CowMotion should only be used after a regular hoof care.


Instruction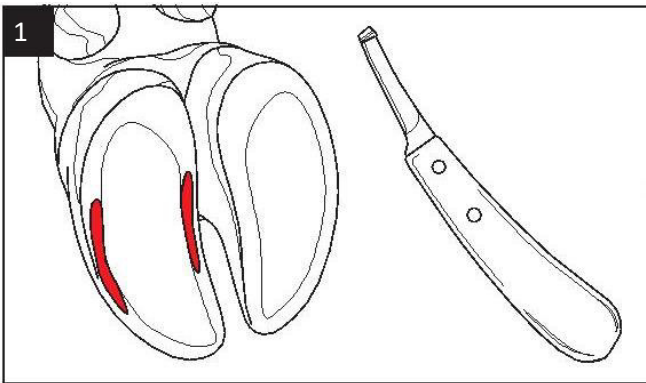


1. Perform claw trimming.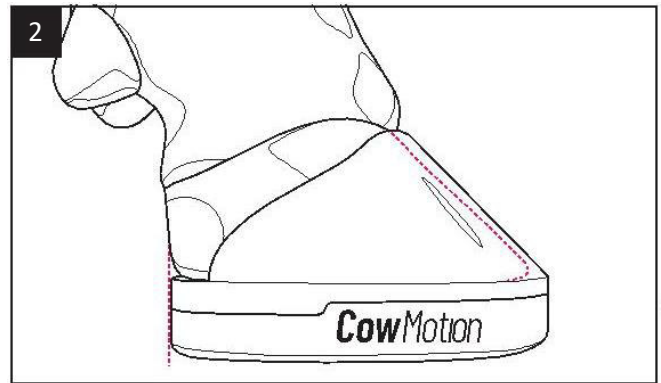


2. Put the orthosis on the claw as a test. The claw should be as flush as possible.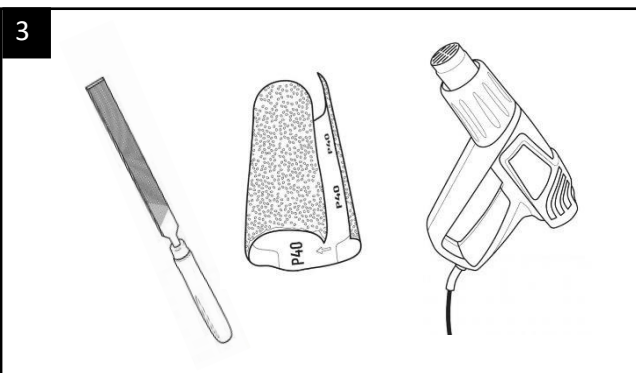

3. Roughen the claw wall with a rasp and the orthosis with the sandpaper in the cap. If necessary dry both with a hot air gun.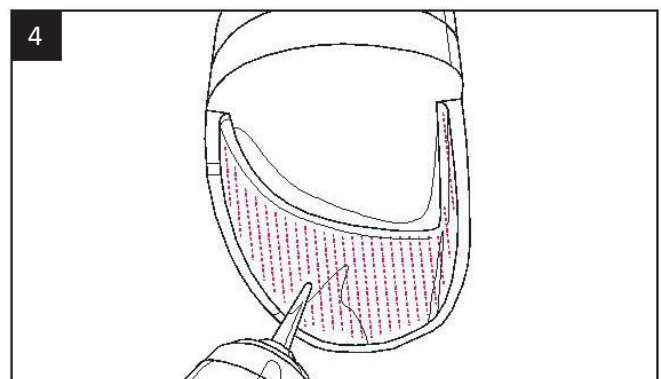

4. Hold the orthosis in your hand with the tip pointing downwards (cap pointing towards the person) Apply the adhesive only to the entire cap (ascending from cap tip to cap end).

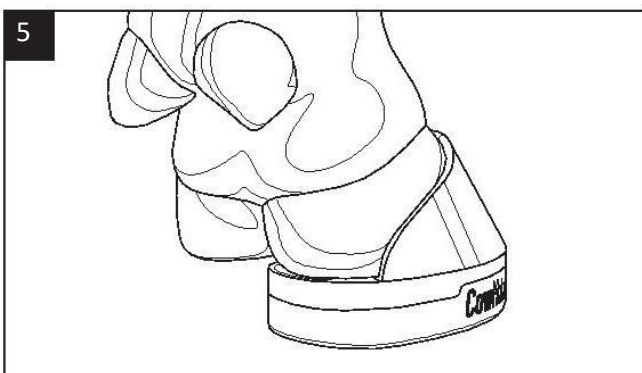5. Immediately apply CowMotion to the claw and hold in position until the glue is hand-tight. (Normally, the orthosis is completely loadable after approx. 5 minutes, please observe the instructions of the adhesive used and the temperature).

CowMotion

CowMotionHoof Glue

Instructions for use:

CowMotion Hoof Glue, black, 10:1, 50ml cartridge

For high-strength, structural and at the same time flexible, durable bonded joints

Speed:	Slow
Processing time:	60-120 seconds*
Curing time:	120-180 seconds*
Consistency:	Thick
Winter suitability:	By tempering. This can be done by placing in warm water or by positioning the cartridge close to the body (e.g. breast pocket).
Summer suitability:	Optimal
Storage temperature:	2 to 20°C
Working temperature	16 to 30°C, below 16°C the adhesive must be warmed up
Warmbox:	34 to 37 °C
Shelf life:	9 - 12 months
CowMotion orthosis:	Ideal
Mixing tip:	Standard mixing tip

*Depends on the processing temperature and especially on the core temperature of the adhesive.

Note! Notice! CowMotion Hoof Glue hardens slowly, has a thick consistency and is ideal for the CowMotion orthosis. Can be used without hesitation in summer. **In winter, it is imperative to ensure the correct storage and processing temperature.**

CowMotion

CowMotion Hoof Glue

Instructions for use:

CowMotion Hoof Glue, black, 1:1, 50ml cartridge

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