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Simple application

CowMotion can be quickly and easily attached to the bovine claw and adjusted. The time in the hoof stand is shortened and reduces the stress factor for the animal. For the farmer, veterinarian and hoof trimmer, this means more efficiency and more relaxed work.

However, it should only be used after regular hoof trimming. The claw must be dry (use a heat gun for this).

Notice!

In principle, our cattle orthoses can be glued with known hoof adhesives. We recommend an adhesive with an open time of 60 seconds so that you have enough time to apply the orthosis and seal it to the cap.

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Innovative construction

CowMotion is made up of two different layers. These ensure optimal transmission to the load-bearing parts of the claw and protect the anatomically sensitive areas. Thanks to the flexible material, the orthosis can be quickly and precisely adapted to any cattle claw.

Tec2Fit GmbH . Holzweg 32 . D-39128 Magdeburg . Germany
+49 391 66279313 . info@cowmotion.de

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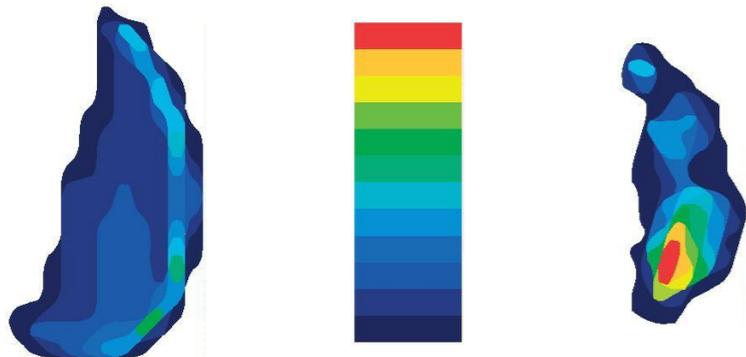
The revolutionary bovine orthosis

CowMotion is the revolutionary orthosis for the treatment of claw diseases and lameness in cattle. It has been developed in close cooperation with veterinarians and claw specialists. CowMotion is the innovative therapy option and, if used consistently, offers decisive advantages for the animal and the owner.

Minimizes the risk of secondary diseases
Relieves pain and ensures sustainable cure
Ensures longer life cycles and fewer departures
Reduces treatment costs

Druckverteilung / Distribution of pressure
CowMotion vs. Holzklötz/Claw block

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Instruction

1. Perform claw trimming.
2. Put the orthosis on the claw. The claw should connect as flush as possible (the tip of the claw can be felt in the tip of the cap). orthosis roughen.
3. Hold the orthosis in your hand with the tip pointing down (cap points to the person)
4. Apply the glue to the entire cap only (ascending from cap tip to cap end)
6. Attach CowMotion to the claw immediately and press for 10 seconds (regularly, the orthosis can be fully loaded after approx. 5 minutes. Please observe the instructions for the adhesive used!)